

SILVER SEAL[®] HYDROGEL DRESSING

Flexible, sterile, non-adherent hydrogel dressings that incorporates the antimicrobial properties of metallic silver coated fiber.



Key Features

- Contours to maintain contact with the wound bed
- Provides a moist environment ideal for wound healing
- Low adhesion, less likely to cause pain during dressing changes
- Helps to control the volume of wound fluid
- Allows for pain-free removal (as often as necessary)

Antimicrobial Power



Releases silver cations for broad spectrum antimicrobial activity against a range of bacteria¹ including:

- ✓ Methicillin-resistant *Staphylococcus aureus* (MRSA)
- ✓ Vancomycin resistant *Enterococcus faecalis* (VRE)
- ✓ *Pseudomonas aeruginosa*

Indications

Indicated for use on:

- **Partial and full thickness dermal ulcers**
- **Leg ulcers**
- **Superficial wounds**
- **Abrasions**
- **First and second degree burns**
- **Donor sites**
- **Debrided and grafted partial thickness wounds**

The SILVER SEAL® Hydrogel Wound Dressing is intended for the management of wounds and to provide an antimicrobial barrier. Topical application of SILVER SEAL Hydrogel Wound Dressing protects the wound area and the construction of SILVER SEAL Hydrogel Wound Dressing permits the passage of wound fluid through the product to be absorbed by an overlying absorbent material. The silver provides effective protection of the dressing against microbial contamination. In vitro studies have shown^{*1} that the silver fiber in SILVER SEAL Hydrogel Wound Dressing is effective in resisting the growth of bacteria.¹

Directions for Use

The SILVER SEAL Hydrogel Wound Dressing should be applied directly to the wound bed after removal of the blue backing (if the wound is wet, also remove the clear plastic topper) then cover with a conventional dressing of choice. The SILVER SEAL Hydrogel Wound Dressing is designed to be left in contact with the wound for not longer than 7 days but may be changed as frequently as desired. Discard all opened but unused dressings.

Do not attempt to re-sterilize.

Contraindications: Ascertained individual hypersensitivity to nylon or silver; cardiac defibrillators.



SILVER SEAL Hydrogel Wound Dressing, 12X Magnification

Sizes Available

Size	Product Code	HCPCS
2in x 3in 5cm x 7.6cm	AB-200300-103	A6242
4in x 5in 10cm x 12.7cm	AB-400500-105	A6243

10 Dressings Per Box

Storage: Store SILVER SEAL Hydrogel Wound Dressings in sealed packaging at room temperature. Avoid excessive heat or humidity.

For external Use Only. Keep out of the reach of children.

**in vitro results do not necessarily correlate with efficacy.*

Reference: 1. Data on file.



For more information please contact
BIONX Surgical at **1-888-472-0030** or
visit **BionxSurgical.com**



SILVER SEAL is a registered trademark of NEXGEL, Inc.
Please refer to the SILVER SEAL® package insert for complete product information.

©2026 BIONX Surgical, All Rights Reserved. SS001-2026